Fototerapia

GINEVRI
TECNOLOGIE BIOMEDICHE
QUALITY FOR LIFE

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Optic Fibre phototherapy lamp BILI LIGHT

NEW

...The challenge of an advanced technology

BILI LIGHT phototherapy lamp uses a new optic fibre technology to achieve:

- A greater uniformity of radiation, concentrating the entire emission onto the patient.
- A much smaller-sized apparatus than conventional phototherapy lamps.

Thanks to the reduced size, it is possible to administer phototherapy directly into the patient's cot, take the lamp from one ward to another or to use it for home-care.

BILI LIGHT can also be used alongside a conventional radiant phototherapy system increasing therapeutic efficiency and reducing exposure time.

The optic assembly is provided with a halogen lamp and a special group of filters which screen the heat and filters ultra violet rays. In this way the radiation emitted has a wave-length between 425 and 475 nm for optimal therapeutic efficiency. The light beam is conveyed to the patient through an optic fibre cable and a pad which can be placed under the patient: **Cable Pad**.

The pad is sealed, waterproof and hygienic. During use the pad is inserted into a disposable protective cover which provides a soft comfortable and hygienic mattress for the patient.

BILI LIGHT lamp is also available with a trolley with pivoting castors and a basket for storing disposable articles, the optic fibre pad, etc.